

Melissa Heavy

Hygiene Mattress



Therapeutic hygiene mattress in 2 layers

Melissa Heavy

Melissa Heavy has been designed for heavy patients up to 250 kg, but is also suitable for people with a normal body weight. Melissa Heavy consists of two layers, one 12 cm layer of highly elastic cold foam 50 kg / m³ at the bottom and a 6 cm layer of soft pillow Flex foam at the top. Ulcer category: 1-2.

The optional mattress covers are fluid proof and hence prevent urine and blood, etc. to penetrate. The hygiene covers are steam permeable and made of a rustle free, 2-way stretch polyurethane fabric. The covers meet the requirements of SS 8760001 and has a zipper with drip edge on 3 or 4 sides (RXI-5) to make it easier to remove them and put them back on when washing them.



Choose between 2 different covers for your mattress. The more simple RX-5 that has no handles or evacuation function or the RXI-5 with handles and a built-in evacuation function-



RX-5



RXI-5



Please read the instruction manual before using the product.



FACTS

| Item No. | Sizes |
|----------|-----------------------|
| 4060 | 200/205 x 80 x 18 cm |
| 4061 | 200/205 x 85 x 18 cm |
| 4062 | 200/205 x 90 x 18 cm |
| 4063 | 200/205 x 105 x 18 cm |
| 4064 | 200/205 x 120x 18 cm |

Mattress weight

Approx. 16 kg depending on length and width (standard sizes).

Warranty – Mattress core

3 years for structural fatigue of the mattress core's B value in accordance with SS 876 00 11.

Warranty – cover

3 year for manufacturing defects. No softener.

Fire standard

The mattress has been tested and meets the requirements of SS 876 00 01, SS-EN 597-1 & 2

User weight: Up to 250 kg

Recycling

The mattress and its cover are fully recyclable and should be sorted as combustible waste. No dismantling is required.

Labelling: Manufacturing year, month, week, GS1 number fulfilled standards and placement in the bed.

Specify choice of cover and desired length according to item number, for example 4062-RXI-5

Results of pressure measurement

| Mattress | Large indenter 300 N | Large indenter 600 N | Small indenter 40 N |
|----------------------|----------------------|----------------------|---------------------|
| Melissa Basic | 38 | 52 | 68 |
| Melissa Parade | 33 | 52 | 67 |
| Melissa Heavy | 32 | 46 | 67 |
| Melissa Flex | 31 | 51 | 62 |

A low value means a low pressure against the body, all values in mm Hg. For heavier users it is especially important that the pressure shown under "large indenter, 600 N" is as low as possible.

FACTS ABOUT PRESSURE MEASUREMENT

To bypass the individual impact from different people, Swerea IVF has developed pressure indenters that simulate the shapes of selected parts of the human body. By applying a load equal to that of a human body of average weight on these indenters, it is possible to objectively evaluate different mattresses and beds using pressure mats. This method of measurement is described in the Swedish standard with the designation SS 876 00 13

Classification of pressure results for mattresses in new condition.

| Load | Extra low | Low | Medium | High |
|----------------------|-----------|-----------|-----------|-----------|
| Large indenter 300 N | <30.0 | 30.0-34.9 | 35.0-39.9 | 40.0-44.9 |
| Large indenter 600 N | <45.0 | 45.0-52.9 | 53.0-60.9 | 61.0-68.9 |
| Small indenter 40 N | <45.0 | 45.0-52.9 | 53.0-60.9 | 61.0-68.9 |



Järven Health Care

Bromsvägen 3 | SE-891 60 Örnsköldsvik | SWEDEN

Telephone: +46 (0)660-22 28 50 | Telefax: +46

(0)660 - 824 10

www.jarven.se | healthcare@jarven.se